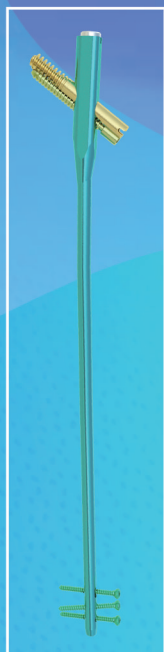
苏南医疗
SUNAN MEDICAL

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Intertrochanteric Antegrade Nail

适应症 Indications

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Simple shaft fractures; Comminuted shaft fractures; Spiral shaft fractures; Long oblique shaft fractures; Segmental shaft fractures Subtrochanteric fractures; Intertrochanteric fractures; Ipsilateral femoral shaft/neck fractures Intracapsular fractures; Nonunions and malunions Polytrauma and multiple fractures; Prophylactic nailing of impending pathologic fractures; Reconstruction following tumor resection and grafting Bone lengthening and shortening

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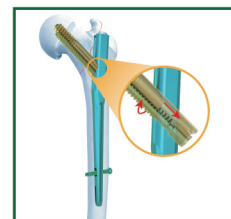
Active linear compression

The active linear compression helps the compression of the femur fracture with rotational control, eliminates excessive sliding reducing the incidence of femoral neck shortening, helps unload the lateral wall and restore the proximal femur.

The clothespin distal tip

The clothespin distal tip of the INTERTAN short nail reduces the overall cross-sectional stiffness of the implant to minimize the risk of iatrogenic fracture during insertion, to ease the actual process of nail insertion and to offer enhanced rotational stability within the proximal femoral canal.

Integrated interlocking screws

The integrated interlocking screws increase stability and resistance to both intra and post-operative fracture rotation about the implant by providing two points of fixation within the femoral neck and head. This directly reduces rotational instability and eliminates the possibility of implant failure due to Z-Effect.

Trapezoidal shape

The smaller cross-section of the INTERTAN nail has the advantage of preserving the Gluteus Medius Tendon while at the same time not compromising the overall strength of the implant. The trapezoidal shape of the nail places more material at the lateral side of the implant where tensile forces are typically the strongest. In addition, the nail's trapezoidal profile also provides enhanced anatomical fit and added rotational stability in the proximal femoral metaphysis similar to that offered by conventional hip stems.

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Add: Kunlunshan road, Hexing, Jinfeng Town, Zhangjiagang, Suzhou, Jiangsu

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Suzhou Sunan Zimmered Medical Instrument co., Ltd.